Compass V5

INSTALL AND GET STARTED

Introduction

Welcome to the Widex® Compass™ fitting software, designed to operate within Noah. Compass offers you an easy way to fit digital and programmable as well as conventional, analogue hearing aids from Widex. This guide explains preconditions, the installation, how you open and close Compass and Compass help.

Before you install

Before you install Compass, you must make sure that your system is ready to use the Compass program, and that you have any extra software needed to use the Compass features.

System requirements

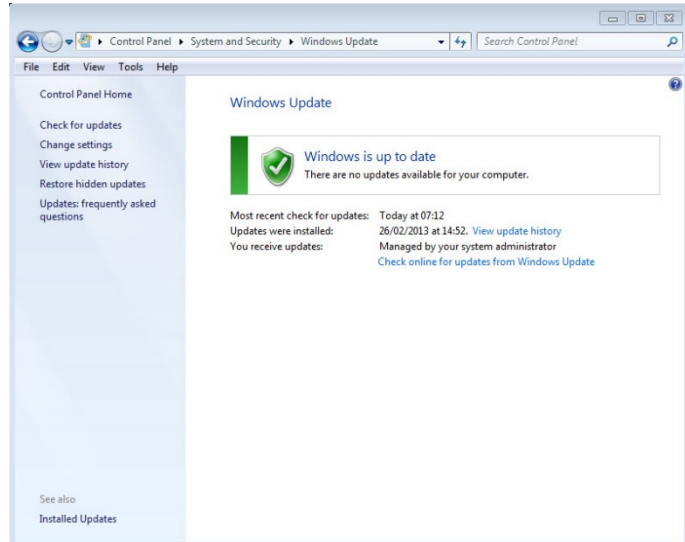
You must make sure that your system fulfils the requirements for the Compass program to run. You can use the *System info* program to help you check that your system fulfils the requirements. The most important requirements are as follows:

Minimum requirement	
Operating system	Windows XP SP3, Windows Vista SP2, Windows 7 SP1, Windows 2003, Windows 2008 & Windows 2012
.NET	If you use the nEARcom programming interface, you must have .NET 2.0.
RAM	1 GB
CPU	1 GHz
Hard drive	500 MB free

Windows Update

Before you install Compass, make sure that your Windows installation is up to date. Do the following to check whether there are updates available for your computer (on Windows 7):

1. Select *Start* in the lower left corner on your screen, and then select *Control Panel*.
2. Select *System and Security* and then *Windows Update*.
3. If updates are available, please install them. It is essential that you install all the updates Microsoft classifies as "Important". Widex urges you to install all updates classified by Microsoft as "Recommended"
4. Restart your computer.

Starting and ending the installation

When you insert the Compass CD-ROM into your CD-ROM drive, the Installation window should open automatically. If it does not, open Windows Explorer, find the CD-ROM drive, and double-click the file *Launch.exe*. Now a window is shown which gives you access to the various elements in a Compass installation.

Due to the large amount of data included in Compass, the program is a 2-CD installation. You can choose between a minimum installation and a full installation. The difference between the two installation types is the amount of data sheets installed. In a minimum installation only the newest data sheets are installed, whereas a full installation installs all data sheets.

Close all open programs on the computer before you install Compass. When you have finished the installation, select the *Exit* item to close the Compass installation program, and restart your computer before starting up Compass.

Ready for Compass program

The Ready for Compass (or System info) program can help you check your system. This program also shows the system requirements.

1. Open Ready for Compass from the Installation window. You can change the language used by selecting a new language from the drop-down list. You can also change the drive checked by the program.
2. If your computer fulfils the system requirements, the message "OK. This computer is ready for Compass V5" is displayed. If the computer does not fulfil the requirements you get a message about this.
3. When you have finished, select *OK* to close the window.

Extra software

The Ready for Compass program also checks whether you have the extra software needed to run Compass. If you need extra software, you can download it from the Internet, or you can contact your WIDEX representative who can help you get it.

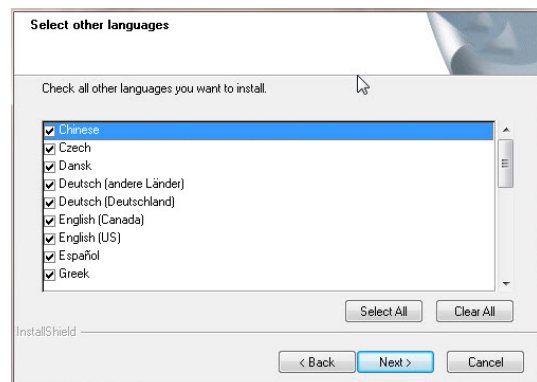
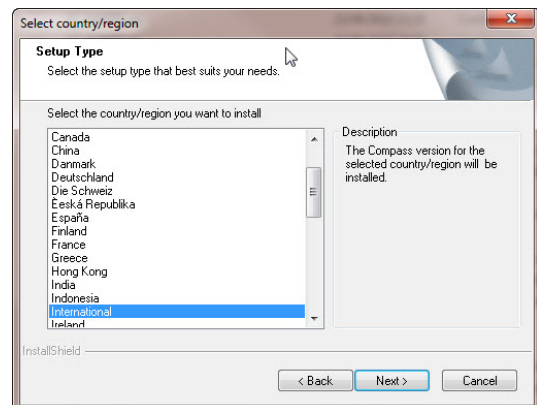
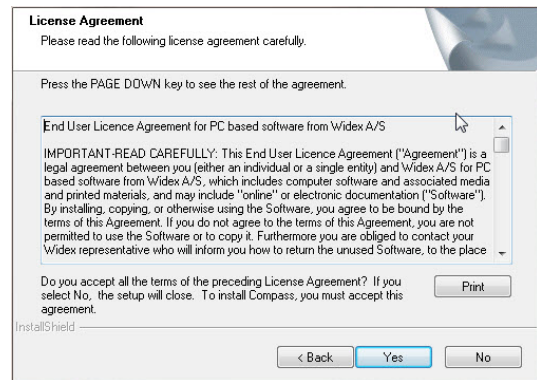
Install Compass

To install Compass, simply follow the instructions in the Installation section below. Please make sure that your Noah software, if any, is closed before you start the installation of Compass. For installation of Noah, please refer to the Noah documentation. If you are working on a network, Compass must be installed and run from each client station in the network.


If you use NOAH 3 or a NOAH 3 OAS, NOAH Engine V3.5.2C (or later) must be installed. We recommend that you use the latest version of Noah 4, and that you keep your Noah installation updated.

Follow the procedure below to install the Compass software.

1. Insert the Compass CD-ROM into your CD-ROM drive. The Installation window, from which you can start the installation, should start up automatically. If it does not, open Windows Explorer, find the CD-ROM drive, and double-click the file *Setup.exe*.
2. Select *Next* in the Welcome window.
3. Read the Licence Agreement. You can print the licence agreement by selecting the *Print* button.
4. Select *Yes* to accept the agreement and move on.
5. Choose whether you want a *Full installation* or a *Minimum installation*, and select *Next*.
6. Select *Next* to use the default destination location. Alternatively, select the *Browse* button and choose another location for the Compass program. Then select *Next*.
7. Select the desired program folder for Compass, and select *Next*.
8. Select your country or region, and select *Next*.
9. Select the main language you want to use in Compass. Compass will use this language when it starts up. Select *Next* to move on.
10. Choose any other language(s) you want to use in Compass. Mark the check box for each language you want to install. You can select all languages or clear the list by means of the buttons below the language list.
11. Select *Next* when you have selected the language(s) you want.
12. Choose whether you want to add a Compass shortcut on the desktop, and select *Next* to move on.

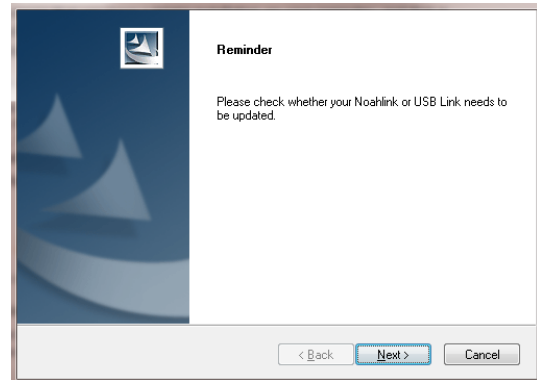13. Check that all information is correct, and select *Next* to begin installing the Compass program. During the installation, you can select the *Cancel* button if you regret installing Compass.
14. When the Compass program has been installed, a dialog box reminds you that your NoahLink or Widex USB Link drivers may need updating. Select *Next*.
15. The installation is now complete. You should restart your computer before you start using Compass. Select *Yes, I want to restart my computer now*. If you want to wait, select *No, I will restart my computer later*.

Note that Compass does not finish the registration procedure until you have restarted your computer. Therefore, it is not recommended to use Compass until after a restart.

16. Select *Finish* to close the installation program. If you selected *Yes* above, your computer automatically restarts.

Open and close Compass

Once you have installed Compass, it is ready for use.

1. Open Noah, and select a client from the database, or enter data for a new client.
2. From Noah, select the Widex fitting module and open it.
3. Compass starts up with the selected client.
4. When you have finished the fitting session, select *Close Compass* from the Compass menu to close Compass again. If you have not already saved your fitting, Compass asks you whether you want to save the fitting in the Noah database before Compass closes.

If you open Compass without opening Noah first, Compass opens in Demo mode, and you can use the features in the software, but you have no access to your client database, which means that you cannot save your fitting data.

Getting help

If you need help with Compass, you have several options, depending on the feature or process you need help with.

- To get a short explanation of the active window in Compass, open the *Using this window* panel at the bottom of the Compass window.

The items in the Compass Help menu take you straight to the help type you select:

- To open the electronic manual, select *Online Manual* from the Compass Help menu, and select the manual you want to open. The manual opens in Adobe Acrobat Reader, and you can navigate the manual and read and print out any information you need from it.
- To open the quick guides, select *Quick Guides* from the Compass Help menu, and then select the quick guide you want to open.
- Furthermore, the program provides tooltips on a lot of the elements on the screen. When you point to one of these elements for a short moment, a tooltip is shown next to it.

Licence information

The Compass software (including any images, photographs, animations, and text incorporated into the software) is owned by Widex A/S. Therefore, you must treat the software like any other copyrighted material.

In case your Compass CD-ROM is defective, please contact your Widex supplier to get a new CD-ROM. If a problem arises in the Compass software, and you need help to solve the problem, you can contact your Widex supplier, or you can send your question to the Widex software support mailbox: wasssupport@widex.com.

This "Start-up guide" is supplied in electronic form. A paper version of this instruction can be requested for no additional cost on <http://widex.pro/gps-startup-guide>, expected delivery time 7 calendar days. The "Start-up guide" is also available on <http://widex.pro/gps-startup-guide>.

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INSTALL AND GET STARTED

Introduction

Welcome to the Widex® Compass™ fitting software, designed to operate within Noah, Compass offers you an easy way to fit digital and programmable as well as conventional, analogue hearing aids from Widex. This guide explains preconditions, the installation, how you open and close Compass and Compass help.

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System requirements


You must make sure that your system fulfils the requirements for the Compass program to run. You can use the *Ready For Compass* program to help you check that your system fulfils the requirements. The most important requirements are as follows:

Minimum requirement	
Operating system	Windows Vista SP2, Windows 7 SP1, Windows 2003, Windows 2008 & Windows 2012
.NET	If you use the nEARcom programming interface, you must have .NET 2.0.
RAM	1 GB
CPU	1 GHz
Hard drive	500 MB free

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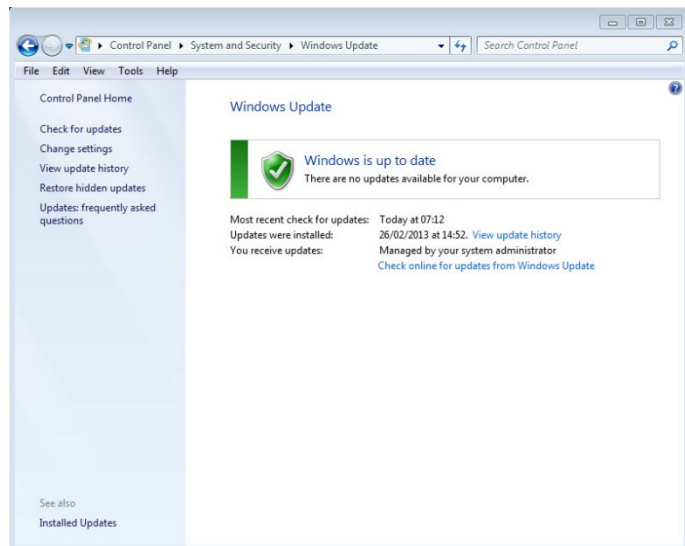Extra software

The *Ready for Compass* program also checks whether you have the extra software needed to run Compass. If you need extra software, you can download it from the Internet, or you can contact your WIDEX representative who can help you get it.

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
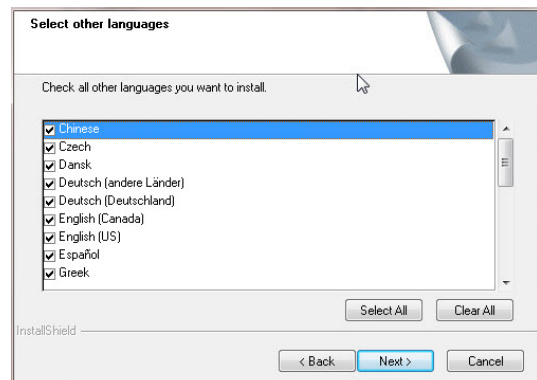
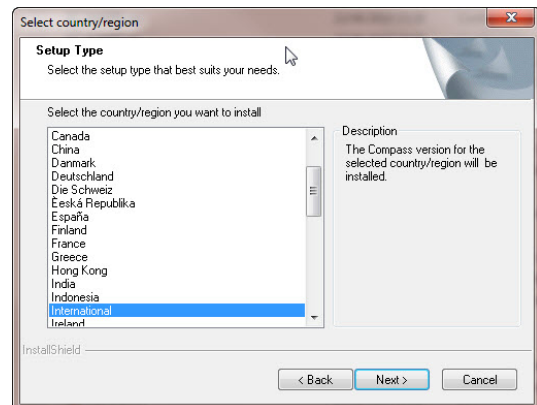
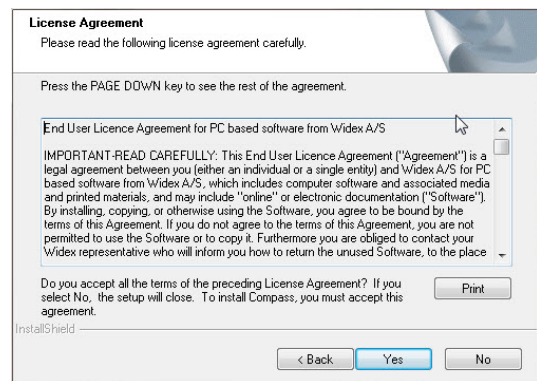
1. Insert the Compass CD-ROM in to your CD-ROM drive. The Installation window, from which you can start the installation, should start up automatically. If it does not, open Windows Explorer, find the CD-ROM drive, and double-click the file *Setup.exe*.
2. Select *Next* in the Welcome window.
3. Read the Licence Agreement. You can print the licence agreement by selecting the *Print* button.
4. Select *Yes* to accept the agreement and move on.
5. Choose whether you want a *Full installation* or a *Minimum installation*, and select *Next*.
6. Select *Next* to use the default destination location. Alternatively, select the *Browse* button and choose another location for the Compass program. Then select *Next*.
7. Select the desired program folder for Compass, and select *Next*.
8. Select your country or region, and select *Next*.
9. Select the main language you want to use in Compass. Compass will use this language when it starts up. Select *Next* to move on.
10. Choose any other language(s) you want to use in Compass. Mark the check box for each language you

want to install. You can select all languages or clear the list by means of the buttons below the language list.

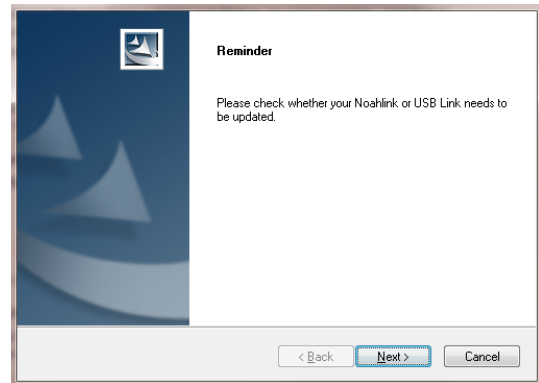11. Select *Next* when you have selected the language(s) you want.
12. Choose whether you want to add a Compass shortcut on the desktop, and select *Next* to move on.
13. Check that all information is correct, and select *Next* to begin installing the Compass program. During the installation, you can select the *Cancel* button if you regret installing Compass.
14. When the Compass program has been installed, a dialog box reminds you that your NoahLink or Widex USB Link drivers may need updating. Select *Next*.
15. The installation is now complete. You should restart your computer before you start using Compass. Select *Yes, I want to restart my computer now*. If you want to wait, select *No, I will restart my computer later*.

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In case, your Compass CD-ROM is defective, please contact your Widex supplier to get a new CD-ROM. If a problem arises in the Compass software, and you need help to solve the problem, you can contact your Widex supplier, or you can send your question to the Widex software support mailbox: wasssupport@widex.com.

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De Compass-software (inclusief afbeeldingen, foto's, animaties en tekst die zijn opgenomen in de software) is eigendom van Widex A/S De software dient dan ook te worden behandeld als ander materiaal met een copyright.